Kalt-Warm-Kompresse

12 x 16 cm / 12 x 29 cm / 20 x 30 cm

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Instructions for use

Indications for cold therapy

- Bruises, sprains, pulled muscles
- Headaches, migraine
- Fever
- Pain relief
- Sunburn
- First-degree burns
- Insect bites
- Haematomas
- Tennis elbow
- Inflammation of muscles and joints*

Application as cold compress

1. The compress is to be cooled in the freezer compartment of the fridge or in a freezer (as low as -25°C). It is ready for use after 1 hour. A compress should always be kept in the freezer compartment so that, in an emergency, it is ready to use at short notice.*
2. Place the compress in the protective cover.
3. Place the compress on the area requiring treatment.

Indications for heat therapy

- Lumbago
- Stomach ache
- Sore muscles
- Back pain
- Menstrual pain
- Tension

Application as hot compress

1. Place the individual compress without the cover in the centre of a microwave oven ensuring it is spread out flat, and heat it.
2. Place the compress on the area requiring treatment.

Indications for heat therapy

- Lumbago
- Stomach ache
- Sore muscles
- Back pain
- Menstrual pain
- Tension

Application as hot compress

1. Place the individual compress without the cover in the centre of a microwave oven ensuring it is spread out flat, and heat it again.
2. Remove the compress from the microwave using oven gloves and carefully knead it to distribute the heat evenly.
3. Place the individual compress without the cover back in the centre of a microwave oven, ensuring it is spread out flat, and heat it again.

Check the temperature of the compress carefully with the palm of your hand: if it is too hot, let it cool down. If the compress is not warm enough, you can heat it for further intervals of 10 seconds at a time. After each 10-second interval, knead the compress and check the temperature. Careful: Do not heat the compress above 50°C, otherwise there is a risk of burns!

Put the cover on the compress and then place it on the area requiring treatment. Do not use the compress without the fabric cover or a cloth. This is not permissible.

General warning

For external use only. Do not use directly on skin. Always put the compress into the cover provided or wrap in a piece of cloth. Do not use on open wounds or damaged skin. Allow children to use under supervision only. Keep out of children's reach. Check product before use and stop using the compress if it becomes brittle or damaged. In such a case please dispose of it in the household waste. Do not apply undue pressure or excess force to the compress. Keep compress away from pointed or sharp objects. If you suffer from circulation problems and other previous illnesses, consult a doctor prior to using the compress.

Additional warnings concerning cold therapy

Unduly long use or use of the cold compress without a protective cover or cloth may result in localised frostbite.

Additional warnings concerning heat therapy

Risk of burns if the product is overheated. Risk of burns if the compress is used without a protective cover or cloth. In the case of defective compresses risk of burns by leaking gel. It is therefore important to check compress carefully for damage or overheating – especially when using it on children. If you suffer from acute or chronic inflammation, consult a doctor prior to using the hot compress. Heat only in the microwave. Protect against open flames. Compress must not be heated by children under the age of 8 and persons with limited physical, sensory or mental abilities or a lack of experience and skill,

unless they are under supervision or have learned how to heat the compress safely and understand the risks involved.

Heating the compress in a microwave may involve a delayed heating process which could even continue after the microwave has been switched off. Therefore, please use the cold compress only after the microwave has cooled to avoid accidental burns, always use oven gloves when removing the compress from the microwave. Heating at higher intensity, longer heating times or higher temperatures than stipulated here can damage the compress. Due to differences in the way microwave ovens are constructed, heating results may vary. To ensure equal distribution of heat within the compress, use exclusively microwave ovens which have a turntable plate.

Contraindications

Do not use in the following cases: restricted sensitivity to temperature, peripheral circulation disorders, Raynaud's phenomenon, when taking painkillers, for children under the age of 3 years, on damaged skin or open wounds. All serious incidents in connection with this product must be reported to the manufacturer as well as the competent authority.

Cleaning the cold/hot compress

Do not use cleaning agents containing solvents such as disinfectants.

Guarantee

This product has been produced according to strict quality guidelines and carefully inspected. The guarantee applies to material or manufacturing defects. Normal wear of the product as well as improper use and damage are excluded from the guarantee. To use the product correctly please strictly adhere to all guidance listed in the instructions for use. The product is intended for private use only and not for commercial use. The gel inside the cold/hot compress is not toxic. In case of contact with the gel, wash with water and soap.

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