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
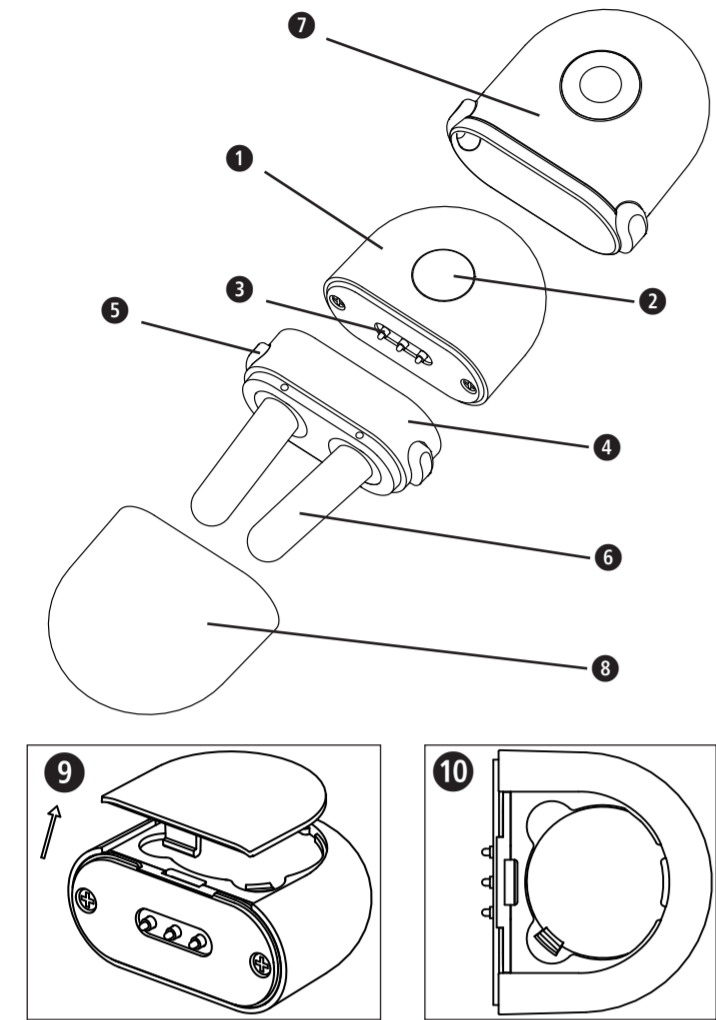
# medisana®
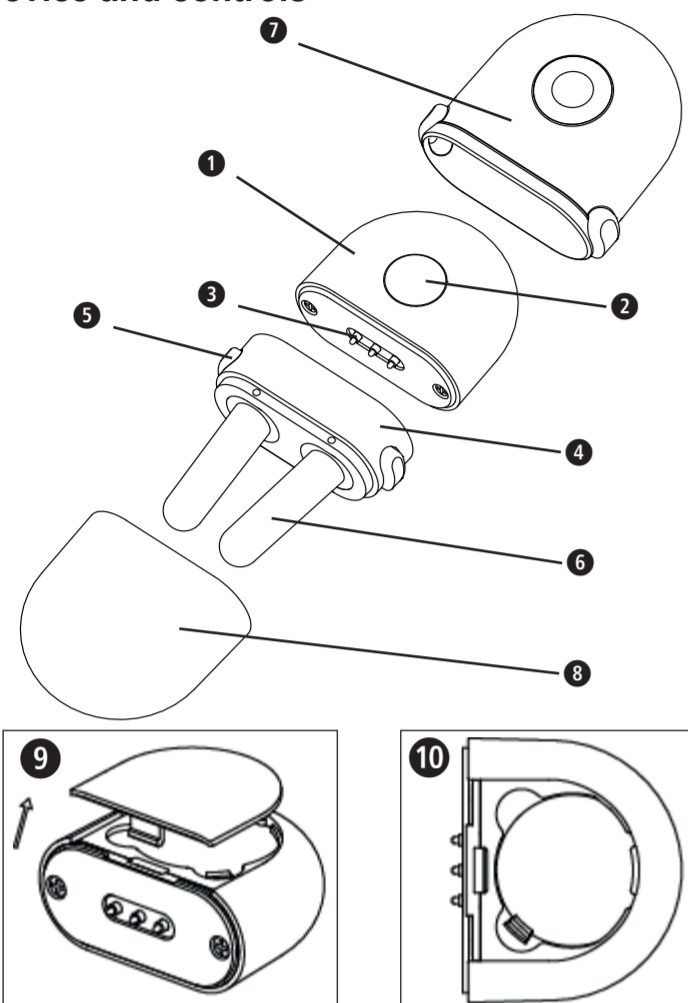
## GB Instruction manual Rhinitis Reliever Medinose Compact

### Device and controls

### Explanation of symbols

This instruction manual belongs to this device. The instruction manual includes important information on the initial start-up and handling. Read this instruction manual completely. Failure to follow these instructions may result in serious injury or damage to the device. Precaution: Please read the instructions for use carefully before use. If you have any questions, please contact customer service or your doctor.

**IP22**

The first characteristic numeral symbol for "Degrees of protection against access to hazardous parts and against solid foreign objects". The second characteristic numeral symbol for "Degrees of protection against ingress of water".

Disposal only according to the "WEEE" Waste Electrical and Electronic Equipment EC Directive.

Recycling symbols/codes: These are used to provide information about the material and its proper use and recycling.

CE 0197

Complies with the European Medical Device Directive (MDD) 93/42/EEC.

REF

Catalogue Number

LOT

Batch code

SN

Serial Number

Device classification: type BF applied part (the nasal probes)

MD

Medical Device

Humidity range

EC REP

Authorized representative in the European Community / European Union

Ambient pressure limitation

Importer

Temperature range

Manufacturer

Date of manufacture

**medisana GmbH,**  
Carl-Schurz-Str. 2, 41460 NEUSS,  
GERMANY

**ANDON HEALTH CO., LTD.**  
No. 3 Jin Ping Street, Ya An Road, Nankai District,  
Tianjin 300190, China  
Tel: 86-22-87611660

EC REP

**iHealthLabs Europe SAS**  
36 rue de Ponthieu, 75008, Paris, France

## GB IMPORTANT INFORMATION! KEEP IN A SAFE PLACE!

Please read this instruction manual carefully, in particular the safety instructions, before using the device. Keep the instruction manual in a safe place for later reference. If you pass the device on to a third party, this instruction manual must remain with the device.

The device must only be used for its intended purpose as described in the instruction manual. Using the appliance for any other purpose invalidates the warranty.

The device is not to be used for commercial use or medical purposes.

- Children under 16 must be supervised by an adult when using this product.
- This device is not suitable for children under the age of 6.
- Keep the device out of the reach of children.
- Never look directly at the light, even when using an optical instrument (for example glasses).
- Do not use the equipment where flammable gas (such as anaesthetic gas, oxygen or hydrogen) or flammable liquid (such as alcohol) are present. Do not use it in an oxygen-enriched environment.
- Do not use the device if you are connected to, or in the vicinity of high-frequency surgical or industrial equipment.
- Do not use the device in the direct vicinity of short-wave or microwave equipment.
- Only use accessories intended for the unit and supplied by the manufacturer.
- You may only clean the device. In the event of malfunctions, do not proceed with any unauthorised repair of the device. Otherwise, this will render any warranty claims void, and will compromise the device safety. Ask your dealer and have repairs done only by **medisana** services.
- Information regarding potential electromagnetic or other interference between the **Medinose Compact** device and other devices together with advice regarding avoidance of such interference, please see the Electromagnetic Compatibility Information. It is suggested that the device be kept at least 30 cm away from other wireless devices, such as WLAN units, microwave ovens, etc. It can't be used near active HF Surgical Equipment and the RF shielded room of an ME System for magneticresonance imaging, where the intensity of EM disturbances is high.
- Please do not share the device with another infective person to avoid cross-infections.
- Stop using the device immediately if you feel any discomfort or pain at any time during the treatment. If the discomfort persists, please seek medical advice.
- Please do not knock down, repair, and rebuild it privately.
- Never immerse the device in water or other liquids.
- If unexpected operation or events should occur, contact the customer service.
- Don't twine the cables around your body, this may cause strangulation.
- Swallowing batteries and/or battery fluid can be extremely dangerous. Keep the batteries and the unit out of the reach of children and disabled persons
- Use of controls or adjustments or performance of procedures other than those specified herein may result in hazardous radiation exposure.
- If you are allergic to plastic, please don't use this device.
- The patient is an intended operator.

## BATTERY SAFETY INFORMATION

- Do not disassemble batteries!
- Clean the battery and device contacts if necessary before putting in the batteries!
- Remove discharged batteries from the device immediately!
- Increased risk of leakage, avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of this parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
- If a battery has been swallowed seek medical attention immediately!
- Replace all of the batteries simultaneously!
- Only replace with batteries of the same type, never use different types of batteries together or used batteries with new ones!
- Insert the batteries correctly, observing the polarity!
- Keep the battery compartment well sealed!
- Remove the batteries from the device if it is not going to be used for an extended period!
- Keep batteries out of children's reach!
- Do not attempt to recharge these batteries! There is a danger of explosion!
- Do not short circuit! There is a danger of explosion!
- Do not throw into a fire! There is a danger of explosion!
- Keep unused batteries in their packaging away from metal objects in order to prevent short circuiting!
- Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!

### Intended use

The Medical Rhinitis Reliever is an innovative and effective treatment for the relief of the symptoms associated with allergic rhinitis like watery eyes, runny nose and sneezing.

### Contraindications:

- The device is safe for all people, with the following exceptions:
- People who are receiving physiotherapy.
- Keep the device out of the reach of children.
- The device is not suitable for use with a pacemaker or other implanted medical devices.
- The device is not suitable for persons with Nasal Polyposis, sensitive skin in the nose or with poor circulation.

### Working Principle

The **Medinose Compact** works by using phototherapy of two different wavelengths. One wavelength will produce small quantities of heat to promote increased blood flow and circulation and the second wavelength (red light therapy) suppresses the cells that release histamine and reduces inflammation, thereby relieving the symptoms of allergic rhinitis. The **Medinose Compact** can be used alone or alongside your normal allergy medication.

### Scope of delivery

Please check first whether the unit is complete and does not show any damage.

If in doubt, do not operate the unit and contact your dealer or service centre.

The scope of delivery includes:

- 1 **medisana** Rhinitis Reliever **Medinose Compact**
- 2 batteries, type CR2032 (3V each)
- 1 protective cover and cap
- 1 instruction manual

If you notice any transport damage when unpacking, please contact your dealer immediately.

Packaging is recyclable or can be returned to the raw material cycle. Please dispose of packaging material that is no longer required properly.

**WARNING**  
Please ensure that the packaging material is kept away from the reach of children! Risk of suffocation!

### Product structure and composition

- 1 Mainframe, with 2 On/Off-button and 3 Connector
- 4 Nose Clamp, with 5 angle adjustment buttons and 6 Nasal Probes
- 7 The Nasal Probes feature LED light emitters
- 7 + 8 Protective cover and cap

## Application

### Insert batteries

- Open the battery cover at the bottom of the mainframe device 1. (See Fig. 9.)
- Load two CR2302 batteries. Please watch out for the polarity. (See Fig. 10.)
- Close the battery cover.

### About the batteries

- When the device makes a buzzing sound five times in a row then the batteries are drained. Replace them with new ones.
- Remove the batteries if the device will not be used for a month or more to avoid relevant damage by battery leakage.
- Rechargeable batteries are not suitable for this device.
- Do not use a new battery together with a used one. Do not use two batteries of different types.
- Avoid the battery fluid to get in your eyes. If it should get in your eyes, immediately rinse with plenty of clean water and contact a physician.
- Make sure the battery cover is intact and not damaged before installing the battery.

### Use

- Stick the Nose Clamp 4 onto the Mainframe 1.
- Insert the two nasal probes 6 into the nose as far as is comfortably possible. Pressing the adjustment buttons 5 on both sides facilitates the insertion and removal of the nasal probe.
- Press the ON/OFF button 2 once to turn the unit on. The buzzer sounds once. The nasal probes 6 start to emit light of two wavelengths.
- One treatment cycle lasts for approximately 3 minutes. The unit will automatically turn off afterwards. If you need to turn the unit off manually during the treatment, press the ON/OFF button 2.
- The **Medinose Compact** can be used three to four times per day. Allow five to six hours between each treatment.

## Anomalies (Troubleshooting)

Issues	Possible causes	Solution
The device cannot be switched on.	The batteries are empty (or not installed).	Install new batteries.
	Batteries are not installed correctly.	Make sure that the batteries are installed correctly. Check the polarity, and if found to be wrong, re-insert the batteries with the correct polarity.
	The On/Off button has not been pressed.	Press the On/Off button 2 to turn on the device.
The buzzer sounds, and the two lights are not lit, or only one is lit.	The Nose Clamp 4 does not fit properly into the Mainframe 1.	Reconnect the Nose Clamp 4 with the Mainframe 1 properly.
The device turns off after 5 buzzer sounds.	The batteries are low.	Replace the batteries with new ones.

## After the treatment is completed / Cleaning

- Remove the batteries if the device will not be used for a month or more to avoid relevant damage of battery leakage.
- This product is not waterproof. Do not clean with any liquid cleaner.
- To clean the unit, use a damp cloth to wipe the surface and nasal probes. Do not allow any liquid to enter the main unit. Make sure it is clean after each use.
- Do not modify this equipment without authorization from the manufacturer. Changes or modifications not approved by the manufacturer will void the user's warranty. Do not disassemble or attempt to repair the device or its components.
- No servicing/maintenance while the device is in use!
- Avoid high temperature and solarization. Do not immerse the device in water as this will result in damage to the device.
- Please wait for approximately 2 hours for the device to warm up or cool down before use when the device is transferred from the ultimate temperature of storage and transportation to the normal operating temperature environment.

## Technical Specifications

Name and model:	<b>medisana</b> Rhinitis Reliever <b>Medinose Compact</b> (model AN-6016)
Power supply:	2 × 3V <sub>≠</sub> , CR2032 alkaline
Battery life:	approx. 30 uses (3 minutes each use)
Technology:	Phototherapy using two different wavelengths (652 nm and 940 nm)
Illuminate position:	Nostrils
Classification:	Internally powered, Type BF applied part, IP22, No AP or APG, continuous operation
Emitting Mode:	Continuous mode
Light power:	3 mW – 6 mW
Operating time:	approx. 3 minutes
Operating environment:	Temperature: 15°C – 40°C (59°F – 104°F) Humidity: 30%RH – 85%RH Atmospheric pressure: 80kPa – 105kPa
Storage and transport environment:	Temperature: -20°C – 60°C (-4°F – 140°F) Humidity: 30%RH – 85%RH Atmospheric pressure: 80kPa – 105kPa
Lifespan:	3 years, under normal operating conditions
Dimensions (L × W × H):	approx. 70 × 39 × 17 mm
Weight:	approx. 21 g
Article number:	45055
EAN-Nummer:	4015588 45055 6
Spare part:	Nose Clamp 4: Art.Nr. 45056, EAN 40 15588 45056 3

## Disposal — Waste Electrical and Electronic Equipment (WEEE)

This product is subject to European Directive 2012/19/EU on waste electrical and electronic equipment and is marked accordingly. Never dispose of electronic devices with household waste. Please seek out information about the local regulations with regard to the correct disposal of electrical and electronic products. Correct disposal helps to protect the environment and human health. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your local authority or your supplier for information about disposal.

## Warranty and repair terms

Your statutory warranty rights are not restricted by our guarantee below. Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is. The following warranty terms apply:

- The warranty period for **medisana** products is 3 years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
- Defects in material or workmanship will be removed free of charge within the warranty period.
- Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
- The following is excluded under the warranty:
  - All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
  - All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - Accessories which are subject to normal wear and tear.
- Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

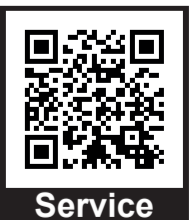
**In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.**

The current version of this instruction manual can be found under <https://docs.medisana.com/45055>.

Information about service can be found here: <https://www.medisana.com/servicepartners>

Manual

Service

**Electromagnetic compatibility information:** See printed instruction manual (also available as a download at <https://docs.medisana.com/45055>).

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